

# Alaska Scientific Crime Detection Laboratory

## DataMaster Check In Form

Issued: 1/13/2017  
Effective: 1/17/2017

Version: DMCIF 2017 R0  
Status: Active

Instrument S/N 100347

Received From Intoximeters

Date 02/23/21

Analyst Derck Walton

1. Reason for instrument return:

*JW 02/23/21*

Returbished by manufacturer, repaired for gas flow errors

2. Change instrument and case location in JusticeTrax Breath Alcohol Database

3. Ensure change in instrument status form is in JusticeTrax

4. Upload

*Instrument was not in the field  
JW 02/23/21*

5. Update agency information and polling status in DM Host Calling Program

6. Check for VOCs

### From Intoximeters

7. Intoximeters Repair Paperwork (attached)

SRO Number: SRO-043866

Received Date: 01/14/2021

Completion Date: 02/10/2021

SRO Type: REPAIR

SRO Description: DMT Preventive Maintenance

Customer ID: C000003651

Customer PO: 01142021-Walton

Contact: Derek Walton

Email: derek.walton@alaska.gov

Ship To:

Alaska Department of Public Safety  
Procurement & Supply Section  
4805 Dr. MLK Jr. Avenue  
Anchorage AK 99507

Units on SRO

1	LEGDMT	LEGACY DMT
---	--------	------------

Service Line: 1		<b>**All Instruments are Calibrated to Factory Specifications**</b>	
<b>Unit Type:</b>	(LEGDMT) LEGACY DMT		
<b>Serial Number:</b>	50100347		
<b>Reason:</b>	DMT Status Messages		
	GasFloEr		
<b>Reason Notes:</b>	Preventative Maintenance, gas flow errors even AFTER regulator replacement		
<b>Resolution:</b>	Replaced		
	RPL comp		
<b>Resolution Notes:</b>	Replaced Detector PCB Assembly and discolored/kinked tubing.		
<b>Parts:</b>	<u>Qty</u>	<u>Part Number</u>	<u>Part Description</u>
	1	90-8000-00	CONTRACT PREV MAINT DMT
<b>Reason:</b>	Preventive Maintenance		
	planned		
<b>Reason Notes:</b>	Maintenance of Filter Wheel Assembly, Sample Chamber, and 5-Way Valve.		
<b>Resolution:</b>	Prev M		
	planned		
<b>Resolution Notes:</b>	Removed filter wheel assembly for cleaning and lubrication. Cleaned & adjusted filter wheel solenoid. Updated filter wheel solenoid mounting screws. Updated stepper motor adjustment lock down screw. Replaced filter wheel motor pinion. Cleaned, lubricated, & adjusted filter wheel moving parts. Removed, inspected, and applied metallic seal tape to mirror blocks of Sample Chamber. Inspected 5-Way Valve.		
	1	001199-00	DMT ELEC REGULATOR V LDO 3.3V OUT 30V IN
<b>Reason:</b>	DMT Functional		
	DigPots		
<b>Reason Notes:</b>	Requires Digital Pots Mod.		
<b>Resolution:</b>	Update		
	DMTVR3		
<b>Resolution Notes:</b>	Removed Control Board and installed TB-30 DMT Digital Pots Mod. All standard factory checks performed. Performed Calibration Adjustment, Wet and Dry Gas Accuracy Checks, Breath Tests, and outgoing Diagnostics Test.		
	1	59-0055-00	DMT ELEC DETECTOR PCB ASSY 41601

# Order Packing Slip Report

**Intoximeters**  
Experience • Service • Integrity

2/17/2021 11:22:40 AM

Packing Slip: 221556

Pack Date: 2/17/2021

SRO: SRO-043866



From: Intoximeters, Inc  
2081 Craig Road  
St Louis MO 63146  
USA

Customer: C000003651

Ship To: 2

Alaska Department of Public Safety  
4805 Dr MLK Jr Ave  
Anchorage AK 99507  
USA

Alaska Department of Public Safety  
Procurement & Supply Section  
4805 Dr. MLK Jr. Avenue  
Anchorage AK 99507  
USA

Customer PO: 01142021-Walton

Ship Via: FDX

FED EX

Weight: 37.00

Delivery Terms : 2D

FOB : ORIGIN

# Packages: 1

Customer Ship Account :

Line Number	Item	UM	Ordered Quantity	Shipment Quantity	Backorder Quantity
1	LEGDMT	LEGACY DMT			50100347
1	90-8000-00	EA		1.00	0.00
	CONTRACT PREV MAINT DMT				
2	001199-00	EA		1.00	0.00
	DMT ELEC REGULATOR V LDO 3.3V OUT 30V IN				
3	59-0055-00	EA		1.00	0.00
	DMT ELEC DETECTOR PCB ASSY 41601				