

Alaska Scientific Crime Detection Laboratory

DataMaster Check In Form

Issued: 1/13/2017
Effective: 1/17/2017

Version: DMCIF 2017 R0
Status: Active

Instrument S/N 100361

Received From Intoximeters

Date 07/12/21

Analyst Dereen Walton

1. Reason for instrument return:

Refurbished by manufacturer, repaired for fatal errors

2. Change instrument and case location in JusticeTrax Breath Alcohol Database

3. Ensure change in instrument status form is in JusticeTrax

4. Upload

*Instrument was not in the field
DMV 07/12/21*

5. Update agency information and polling status in DM Host Calling Program

6. Check for VOCs

From Intoximeters

7. Intoximeters Repair Paperwork (attached)

SRO Number: SRO-044605

Received Date: 02/25/2021

Completion Date: 06/11/2021

SRO Type: REPAIR

SRO Description: DMT Repair/Preventive Maintenance

Customer ID: C000003651

Customer PO: 021221-Walton

Contact:

Email:

Ship To:

Units on SRO

National Patent
2090 Harrington
Mansfield OH 44907
USA

1	LEGDMT	LEGACY DMT
---	--------	------------

Service Line: 1	**All Instruments are Calibrated to Factory Specifications**	
Unit Type:	(LEGDMT) LEGACY DMT	
Serial Number:	50100361	
Reason:	Preventive Maintenance	
	planned	
Reason Notes:	Preventive Maintenance	
Resolution:	Prev M	
	planned	
Resolution Notes:	<p>Removed filter wheel assembly for cleaning and lubrication. Cleaned & adjusted filter wheel solenoid. Replaced filter wheel motor pinion. Cleaned, lubricated, & adjusted filter wheel moving parts. Applied epoxy to stepper motor wires. Removed and cleaned Sample Chamber. Applied metallic seal tape to mirror blocks and cleaned hazy chamber windows. Replaced discolored tubing and inspected 5-Way Valve.</p> <p>Removed Control Board and installed TB-30 DMT Digital Pots Mod. Optimized detector position. Adjusted Tech Screen settings. Verified Modem with laptop and DMT Tech Software using phone cord.</p>	
Parts:	<u>Qty</u>	<u>Part Number</u>
		<u>Part Description</u>
	1	99-0001-00 MISC REPAIR
Reason:	DMT Status Messages	
	DetOverF	
Reason Notes:	Customer Reported Issue: frequent fatal errors – may be related to modem board	
Resolution:	Replaced	
	RPL comp	
Resolution Notes:	Replaced leaking breath tube, erratic touch screen, chopper motor assembly, detector terminal block on control board, and rubber bumper feet.	
	1	90-8000-00 CONTRACT PREV MAINT DMT
	4	59-0085-00 DMT HRDW BASE RUBBER BUMPER 31360
	1	59-0016-00 DMT TUBE ASSY BREATH 12VDC 42222
	1	59-0101-00 DMT ELEC MEMBRANE 8.4IN TOUCH REPL 37553
	1	59-0006-00 DMT HRDW ASSY MOTOR 12X20.6MM 41310
	1	59-0132-00 DMT ELEC SCREW TERMINAL 7 POS 28528

Order Packing Slip Report

Intoximeters
Experience • Service • Integrity

6/16/2021 2:23:55 PM

Packing Slip: 230496

Pack Date: 6/16/2021

SRO: SRO-044605



From: Intoximeters, Inc
2081 Craig Road
St Louis MO 63146
USA

Customer: C000003651

Ship To: 1

Alaska Department of Public Safety
4805 Dr MLK Jr Ave
Anchorage AK 99507
USA

Alaska Scientific Crime Detection
Laboratory
4805 Dr MLK Jr Ave
Anchorage AK 99507
USA

Customer PO: 021221-Walton

Ship Via: FDX

FED EX

Weight: 38.00

Delivery Terms: 2D

FOB: ORIGIN

Packages: 1

Customer Ship Account:

Line Number	Item	UM	Ordered Quantity	Shipment Quantity	Backorder Quantity
1	LEGDMT - LEGACY DMT				50100361
1	90-8000-00	EA		1.00	0.00
	CONTRACT PREV MAINT DMT				
2	59-0085-00	EA		4.00	0.00
	DMT HRDW BASE RUBBER BUMPER 31360				
3	59-0016-00	EA		1.00	0.00
	DMT TUBE ASSY BREATH 12VDC 42222				
4	59-0101-00	EA		1.00	0.00
	DMT ELEC MEMBRANE 8.4IN TOUCH REPL 37553				
5	59-0006-00	EA		1.00	0.00
	DMT HRDW ASSY MOTOR 12X20.6MM 41310				
6	59-0132-00	EA		1.00	0.00
	DMT ELEC SCREW TERMINAL 7 POS 28528				